

## Calendar clock with reminders

CARY Base

CARY Base is a calendar clock for people who need extra help keeping track of the hours of the day, as well as the day's events.

CARY Base shows information about the date and time, as well as whether it is morning, daytime, evening or night. By pressing the screen, the information will be read out loud.

The day's events can be shown on the screen, and spoken reminders can be received at desired times. Family members or caregivers can help to keep your calendar up-to-date remotely.


Art.nr: 508900



**ABILIA**

## USING CARY BASE

Place CARY Base in a spot where the user is frequently present. For example, this could be the bedroom, at the TV, or in the kitchen. Connect to the Internet and the myAbilia web service if family members and caregivers need to keep the daily plan updated remotely.

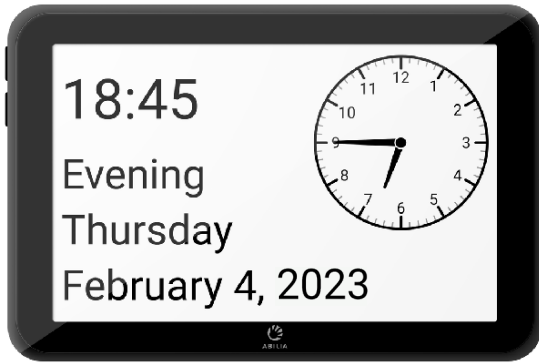

## CUSTOMISING CARY BASE


Set what should be shown on the screen according to the user's needs. Remember to set the CARY Base so that the screen shows "night" when the user should sleep, and "morning" when the user gets up.

Instead of pressing the screen, an external switch (accessory) can be used to have the information read out loud.




## GET AN OVERVIEW

Use the day plan in CARY Base to show the most important events of the day.

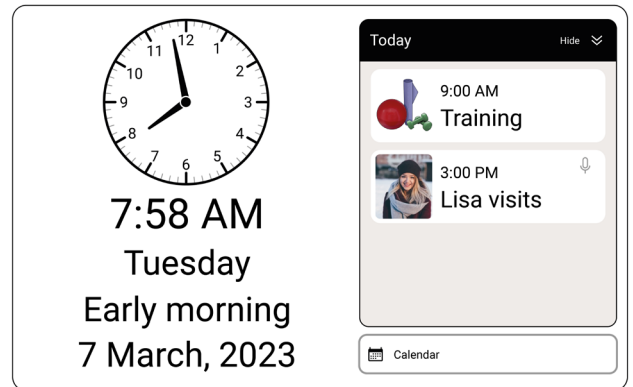

## GET REMINDERS

If desired, CARY Base can emit a signal when an event begins. At the same time, a voice message can also be played.

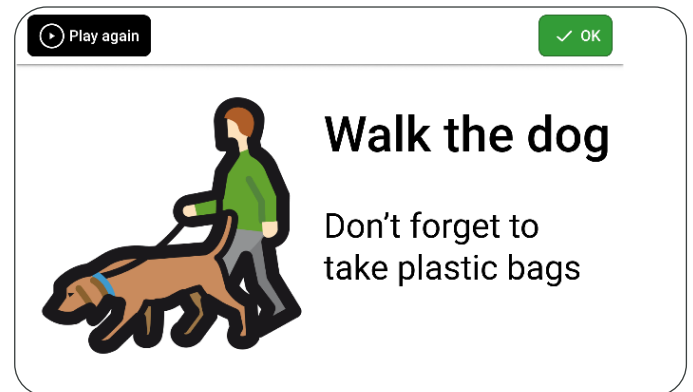

CARY Base is a medical device intended for indoor use in a home environment.

### TECHNICAL

Dimensions	207 x 137 x 74 mm
Weight	520 g
Screen	8 inches
Available languages	English (British or Australian), Swedish, Norwegian, Danish, Finnish, Sami, Icelandic*, German, Dutch, Flemish, French, Spanish, Italian, Slovenian*, Bosnian*, Croatian*, Serbian*, Czech, Polish, Albanian*, Arabic, Persian*, Turkish, Somali*, Hungarian*, Russian, Greek (*= display reader not available).